




# Bio Tuner Lite

## Troubleshooting

Models: BTL1

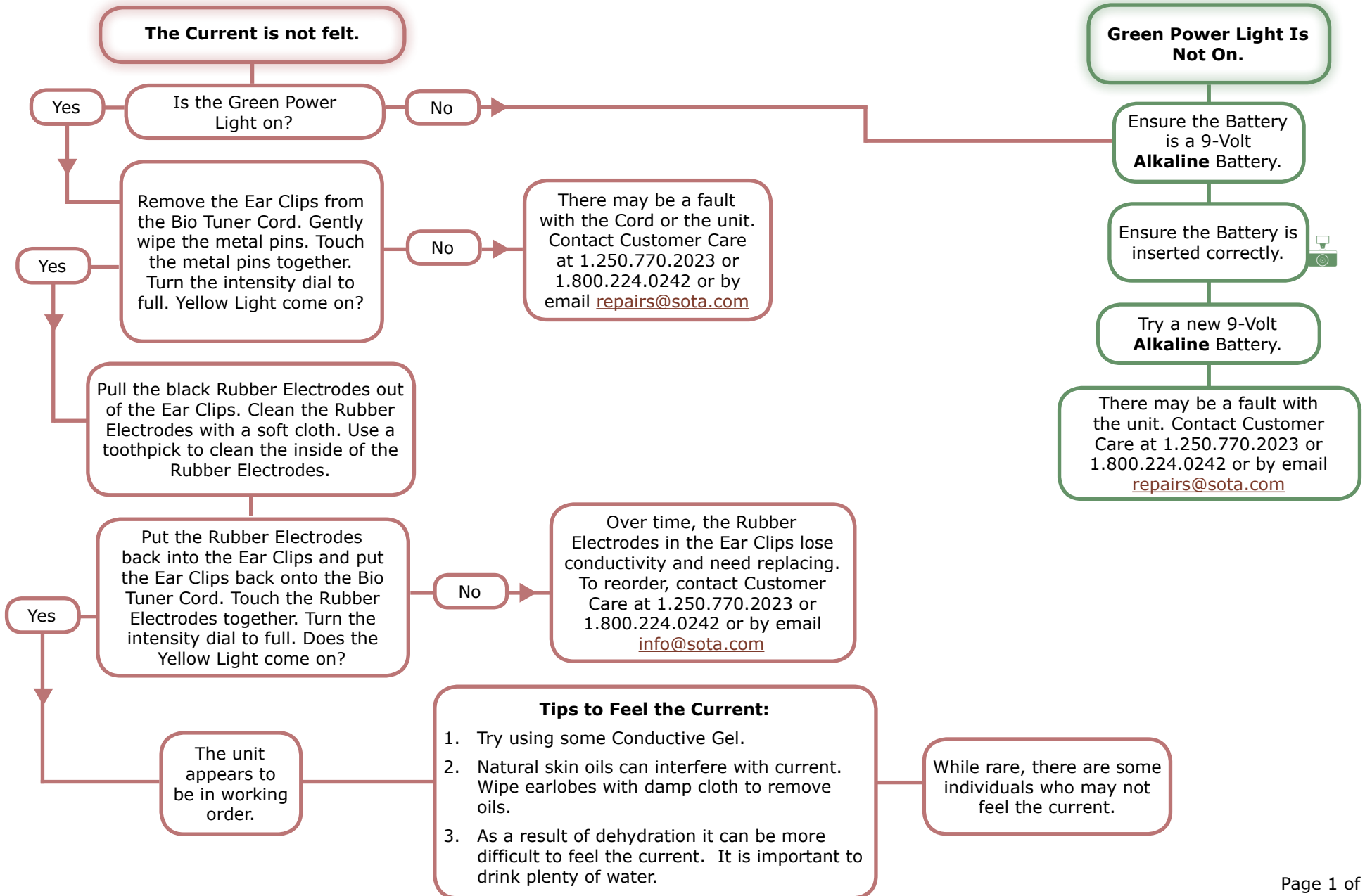



# Bio Tuner Lite Troubleshooting

Models: BTL1

**The Yellow Indicator Light does not come on.**

Yes

Can you feel the current?

No

The unit is working fine. The Yellow Indicator Light is used for testing to ensure the Unit, Cord and Ear Clips are working correctly. It does not necessarily come on when using the unit.

See  
**The Current Is Not Felt.**

**Ear Clips sting when placed on earlobes.**

Yes

Check for full conductivity by touching the Ear Clip Rubber Electrodes together. Turn the intensity dial to full. Does the Yellow Light come on brightly?

No

Natural skin oils can interfere with the current. Wipe ear lobes and electrodes with damp cloth to remove oils.

Ensure the Ear Clips are adequately moistened.

Try applying a Conductive or Aloe Vera Gel to the ear lobes.

Over time, the Rubber Electrodes in the Ear Clips lose conductivity and need replacing. To reorder, contact Customer Care at 1.250.770.2023 or 1.800.224.0242 or by email [info@sota.com](mailto:info@sota.com)

**The current is felt more on one side than the other.**

This is not a fault, it is common to feel the current more on one side than the other.

Over time, the Rubber Electrodes in the Ear Clips lose conductivity and need replacing. To reorder, contact Customer Care at 1.250.770.2023 or 1.800.224.0242 or by email [info@sota.com](mailto:info@sota.com)